

CORONAVIRUS STUDENT SUPPORT GUIDE

To get the best out of your Roger Touchscreen we recommend the following:

| TIP | DESCRIPTION |
|-------------------------------------|---|
| USE A MASK | Using a mask can affect clarity of microphones used for listening or captioning. Results can vary between masks. Clear masks may help better lip reading but impact sound quality. Using a Roger Touchscreen will improve clarity and accessibility. |
| CONNECT YOUR DEVICE DIRECTLY | Connect your touchscreen to your iPad / Laptop device for a direct feed while using video conference technology. See <i>“Connecting your Laptop”</i> or <i>“Touchscreen Cheat Sheet”</i> . |
| CAPTIONING | Captioning can be used to help overcome communication barriers. Turn on captioning when ever possible. |
| SOUNDFIELD | Use a Soundfield to enable the teacher’s voice to be evenly distributed over a classroom whilst maintaining social distancing (The numbers of people and the size of room may impact range). |
| CLEANING YOUR EQUIPMENT | Please clean all Roger equipment between users with alcohol free wipes, no sprays or liquids, as per Phonak Guidelines. |
| LIMIT SHARED EQUIPMENT | Limit the use of shared equipment that could harbor germs. eg. Lanyards, Microphones & Headphones. Disposable headphone covers are available |



Hearing Loop Australia
Keeping your audience in the loop

www.hearingloop.com.au
info@hearingloop.com.au
1300 669 721