

ROGER WALL PILOT CHEAT SHEET

CONNECTING THE WALL PILOT TO ROGER TOUCHSCREEN

Option 1

- 1.1 Turn on Wall Pilot.
- 1.2 Turn off the Touchscreen for at least 3 seconds, then whilst holding the Touchscreen close to Wall Pilot, turn it on.
- 1.3 "Connected" will appear on Touchscreen when the devices are paired and a  will appear on the display of the Wall Pilot.



Option 2

- 2.1 Both the Roger Touchscreen and Wall Pilot must both be turned on to connect
- 2.2 Hold Roger Touchscreen Mic within 5cm
- 2.3 From the Home page on the wall pilot press 'OK' button
- 2.4 Use arrow and 'OK' button to select the following options. Select "Network", "Join Net" & "Yes".



COMPATIBLE DEVICES

Roger Neckloop



Hearing Aid with Roger™ X (02) Receiver or iN Receivers



Cochlear Implant with Mini Mic 2 Processor & Roger Receiver



CONNECTING DEVICES TO THE NETWORK USING THE WALL PILOT

Compatible devices with automatically connect to the Roger Network in the room when it passes within 2m of the Wall Pilot

Tip: Connect the Touchscreen to the Wall Pilot first.



Hearing Loop Australia
Keeping your audience in the loop

www.hearingloop.com.au
info@hearingloop.com.au
1300 669 721