

HOW TO USE ROGER™ RECEIVER

Your hearing system consists of three main elements: Roger MyLink, your hearing aids and a Roger transmitter, such as Roger Touchscreen or Roger Media Hub. The Roger microphone transmits the audio or speaker's voice wirelessly to your hearing aids via the Roger MyLink. You only need to activate the T-Coil in your hearing aid (s) to hear the speaker's voice loud and clear in your ear(s).

PLEASE SEE ROGER TOUCHSCREEN CHEAT SHEET OR ROGER MEDIA HUB CHEAT SHEET FOR INSTRUCTIONS ON CONNECTING / USING THE TRANSMITTER



Roger MyLink (02) - Use with T-Switch / Telecoil



1. Charging Socket
2. 2.5mm Headphone




1. For hearing devices fitted with telecoil - put neck loop around your neck. It can be tucked underneath clothing.
2. Switch receiver MyLink **ON** with the Green side visible.
3. Switch hearing device to "telecoil" mode.
4. Adjust loudness on Roger™ receiver & hearing devices to desired volume.

- ① Volume control
- ② On/off



Roger MyLink (02) - Use with Headphones

1. For hearing devices fitted with telecoil - put neck loop around your neck.
2. Switch MyLink receiver **ON**.

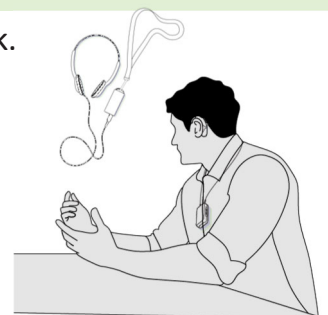

Please return the receiver and attachments at the end of the program.