


ROGER SOUNDFIELD SPEAKER VERSION 2

5000 & 7000 Speakers Version 2

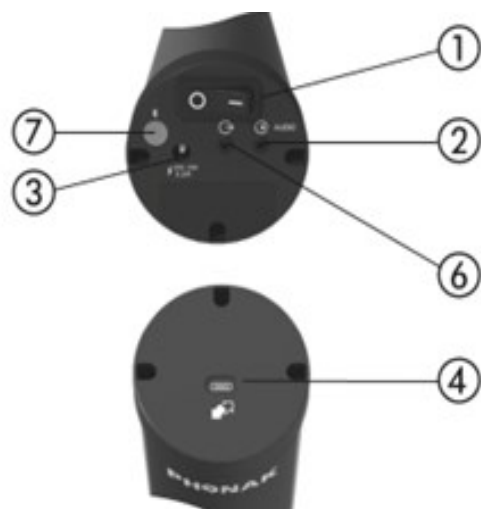

- ① On/off
- ② 3.5 mm audio input
- ③ Power socket
- ④ USB C
- ⑤ Indicator light (LED)
- ⑥ 3.5 mm audio output
- ⑦ Bluetooth

How to connect Bluetooth feature


- Press Shortly the Bluetooth Button on the digimaster , as circled above.
- LED: Fast blue blinking (60s pairing mode)
- Pair device (BT settings : Digimaster xxxx)
- LED: connected, blinking blue twice (1s)

Indicator Lights



- Roger DigiMaster off
- Standard operation mode
- Roger Microphone off/out of range
- Muted
- Volume +/-
- Software update
- Call technician

Audio Connections

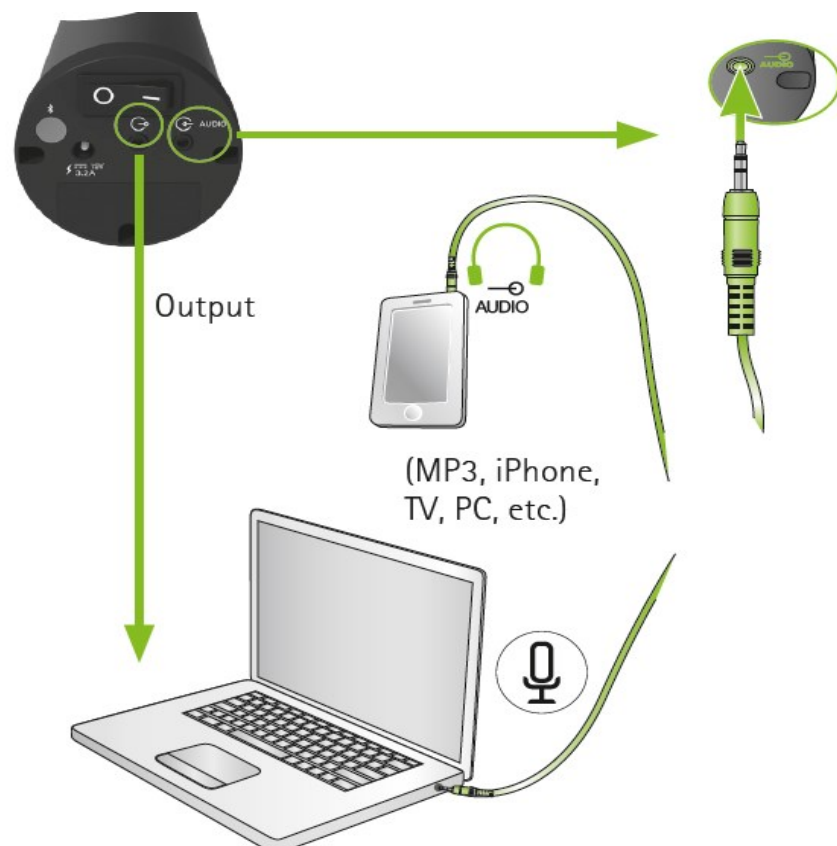

Soundfield Stands			
Soundfield Tripod Stand	Soundfield Wall Mount	Soundfield Floor Stand	Soundfield Desk Stand