


ROGER SELECT V3 CHEAT SHEET


Turn On

Press and hold on/off button for at least 3 seconds until you get green flashing light on device.

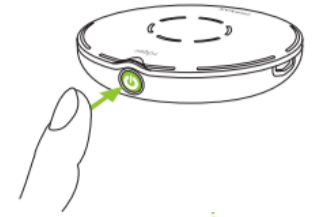

Connect to Hearing Aid Receiver / Roger Neckloop

Press the link/connect button on the back of the Select and hold within 5 cm to either the In-built Roger X receiver in the hearing aid or the Roger Neckloop. The person will hear beep upon connection.

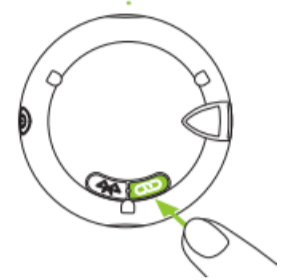

Using the Roger Select on a table or flat surface


The Roger Select V3 has 6 microphones that can be configured in different ways when **using it on a flat surface or held in a horizontal position.**



To use the Roger Select V3 in a general/omnidirectional mode, which enables the listener to hear all speakers around the table, press the centre button. The Roger Select V3 will automatically focus in on a speaker's voice. Green lights will be displayed around the edge of the Roger Select V3.




To manually choose which microphone on the Roger Select is activated, which enables the listener to focus on a specific speaker(s) voice around the table, press the appropriate section. Green lights will only be displayed around the Roger Select V3 where the microphone is activated. To revert back to general mode, press the centre button.

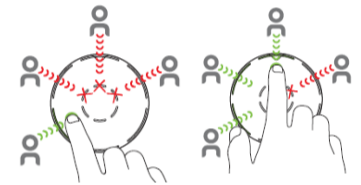


Using the Roger Select V3 for an individual speaker

The Roger Select V3 can be used in a single directional mode for an individual speaker by holding it in a vertical direction. When held vertically, the Roger Select V3 **can only be used in single directional mode.** The Roger Select V3 can be attached to a lanyard to enable it to be worn around the person's neck.

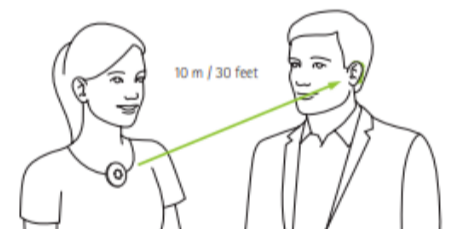

Battery status

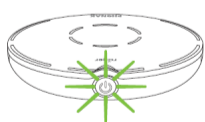

Green fast blinking	Charging
Green solid	Fully charged
Green slow blinking	Switched on, battery full
Orange blinking	Battery half full
Red blinking	Battery empty, charge Roger Select

Tips:

- Refer to the User Guide for device specific information
- When connecting to a telephone or digital TV, an adaptor may be required. Refer to Troubleshooting guides or contact phone support for advice.

Troubleshooting Tips!

I cannot hear from my Roger microphone

- 1** Your Roger microphone is muted (red LED)
Unmute your Roger microphone by pressing the center touch key
- 2** Wrong direction is selected
Tap on the segment that is in the direction of the talker
- 3** Your Roger receivers are not connected to your Roger microphone
Hold your Roger microphone close to each Roger receiver or Roger compatible hearing aid and press the Connect button
- 4** Your Roger receiver is not working
Make sure you have attached the Roger receiver to your hearing aid and that it is powered on
- 5** Your hearing aids are not set to the correct program
Check that your hearing aids are set to the correct program (Roger/FM/DAI/EXT)