

ROGER SELECT V3 PORTABLE KIT USER GUIDE

Kit Contents

The Roger Select V3 Portable kit includes the Roger Select V3, Charging Rack with 2 neckloop adaptors, 2 Roger Neckloops and 2 replaceable headphones all in a sturdy portable case. The Roger Select V3 Portable kit provides a flexible hearing assistance solution that can be used with any person who is hard of hearing. The Roger Neckloops can be used in conjunction with the telecoil program in hearing aids or by wearing headphones. **Ensure equipment is charged before each use.**




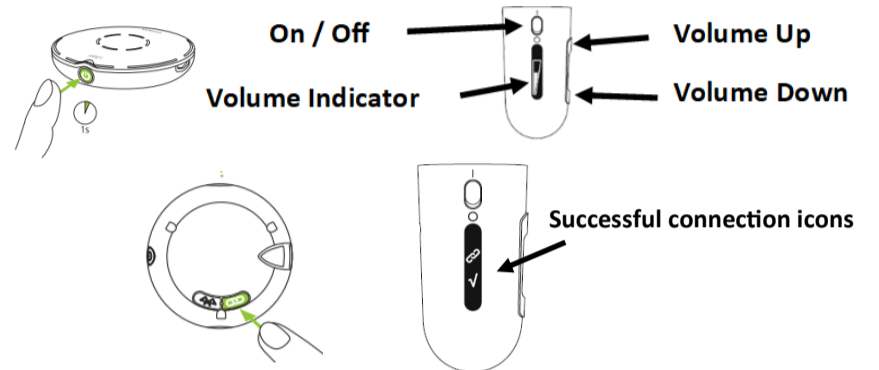
Pairing the Roger Select V3 and Roger Neckloop

Turn on both the Roger Select V3 and Roger Neckloop;

Roger Select V3 —press and hold the on/off button until the device flashes green


Roger Neckloop—move the on/off switch to the on position

Place the two devices side by side. Press the Connect Button on  the Roger Select. A tick icon will appear on the Roger Neckloop once successfully paired with the Roger Select.



Roger Neckloop—using with telecoil program in hearing aids

1. Switch on the Roger Neckloop.
2. Put the Roger Neckloop around the neck. It can be tucked underneath clothing.
3. Switch hearing device to telecoil mode.
4. Adjust loudness on the Roger Neckloop and/or hearing devices to desired volume.

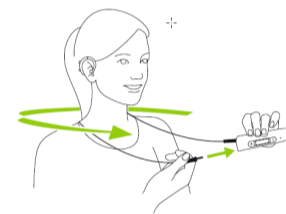

Roger Neckloop—using with headphones

1. Switch on the Roger Neckloop.
2. Connect headphones to the side of the Roger Neckloop and adjust loudness to desired vol-




Roger Select V3 — using on a table or flat surface

The Roger Select V3 has 6 microphones that can be configured in different ways when **using it on a flat surface or held in a horizontal position.**


To use the Roger Select V3 in a general/omnidirectional mode, which enables the listener to hear all speakers around the table, press the centre button. The Roger Select V3 will automatically focus in on a speakers voice. Green lights will be displayed around the edge of the Roger Select.

To focus in on a specific speaker(s) voice, manually choose the microphone on the Roger Select V3 closest to the speaker. Green lights will only be displayed around the Roger Select V3 where the microphone is activated. To revert back to general mode, press the centre button.

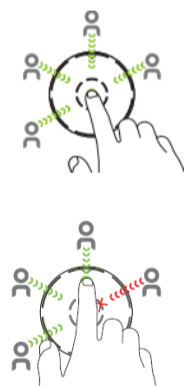


Roger Select V3 — using for an individual speaker

The Roger Select V3 can be used in a single directional mode for an individual speaker by holding it in a vertical direction. When held vertically, the Roger Select V3 **can only be used in single directional mode.** The Roger Select V3 can be attached to a lanyard to enable it to be worn around the person's neck.

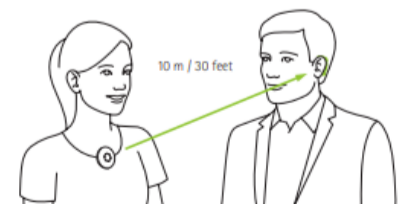

Battery status

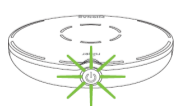

Green fast blinking	Charging
Green solid	Fully charged
Green slow blinking	Switched on, battery full
Orange blinking	Battery half full
Red blinking	Battery empty, charge Roger Select



Hearing Loop Australia
Keeping your audience in the loop

www.hearingloop.com.au
info@hearingloop.com.au
1300 669 721