

ROGER TOUCHSCREEN V3 CHEAT SHEET

Getting Started

Turn on.

Press and hold on/off button for at least 3 seconds until the green flashing light on the Touchscreen. Swipe the screen of the Touchscreen left or right to access the menus.



Connect to Soundfield Speaker

Turn the Soundfield Speaker on using the switch at the bottom of the speaker. A blue light will flash at the top of the speaker.

Hold the Touchscreen V3 within 5 cm of the blue light at the top of the Soundfield Speaker. Press Connect on the front screen of the Touchscreen V3. The Soundfield Speaker light will be solid blue when connected.



Connect to Hearing Aid Receiver / Neckloop (if required)

Hold the Touchscreen within 5 cm of either the In-built Roger Receiver, Roger X Receiver or Roger Neckloop. Select Connect on the front screen of the Touchscreen V3. The student will hear a beep upon connection.

To check what devices are connected to the Touchscreen V3, select List Devices. The connected devices within 5cm will be displayed.



Devices will remain connected to the Touchscreen V3 until the network is reset by selecting New Net, or the device is connected to a different transmitter / network.



Begin Teaching / Talking

Connect the lanyard to the Touchscreen V3 and place the Touchscreen V3 around the neck and shorten the lanyard by pulling the strap so the device is within 15cm of the mouth. Begin talking.

Press the Mute button to activate / deactivate the microphone as required.

The Mute button light will be solid green when transmitting and flash red when muted. The light will be blue if the Touchscreen is the secondary mic in a team teaching network.




Team Teaching

Multiple Touchscreen V3 microphones can be connected together to allow the voice of multiple talkers to transmitter to Roger Receivers and the Speaker—one 5000 V2 or one/multiple 7000 V2 speakers.


The Touchscreen V3 that is connected to the Speaker and other transmitters such as the Roger Media Hub or the Roger Pass-around mic, is the Primary microphone. To connect additional Touchscreen V3 microphones, hold the additional Touchscreen V3 within 10cm of the Primary Touchscreen V3. Select the Connect on the Primary Touchscreen V3. The connected Touchscreen/s will be Secondary microphones.

The Primary Touchscreen V3 will display a **1** and the Secondary Touchscreen V3(s) will display a **2** in the top left corner of the display screen.

Once finished team teaching, hold the Secondary Touchscreen V3 next to the Primary Touchscreen V3 and select Disconnect  on the Primary Touchscreen V3.



Roger Receivers

The Touchscreen V3 has an unlimited number of Roger Receiver licences that can be loaded into hearing aids which have Roger Direct enabled. This is done by turning the hearing aid off, turning it back on and laying it next to the Touchscreen V3. Select Roger Direct  on the Touchscreen V3. A tick will appear once the Receivers are loaded into the hearing aid. The Touchscreen V3 can now be connected to the hearing aid as above. For more information refer to the *Roger Direct loading Roger Receivers* flyer.

